



**STRIPS NEED BOTH OPEN
AND EXPIRATION DATES
THEY ARE GOOD FOR
6 MONTHS ONCE OPENED**



**QC NEEDS BOTH OPEN AND
EXPIRATION DATES
THEY ARE GOOD FOR
3 MONTHS ONCE OPENED**

RUNNING QC

- Always hold the meter level or pointed slightly downward with the test strip when dosing.
- Proper orientation prevents control solution from running down the strip into the meter.
- Allowing liquid to run into the meter may damage the meter!



**QC RUN WAS
SUCCESSFUL**

**METER IS
NOT LOCKED**



**QC RUN WAS NOT
SUCCESSFUL**


**METER IS STILL
LOCKED**

**LEVEL 1 AND LEVEL 3
MUST PASS ONE AFTER
THE OTHER**

DOCKING THE METER

**EACH TOTE HAS 5 BATTERIES THAT
ARE NEEDED FOR THE WI-FI RADIO**

ONCE METER IS
DOCKED LOOK FOR
THE YELLOW
TO BE ON

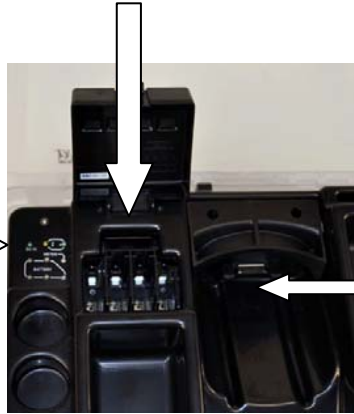

**BE SURE TO DOCK THE
METER FIRMLY SO THAT IT
MAKES CONTACT WITH THE
TOTE**



**THIS IS WHAT THE
METER LOOKS LIKE
WHEN IT HAS
CONNECTED
SUCCESSFULLY**



**THE YELLOW TRIANGLE IS AN
INDICATOR THAT THE METER
TRANSFER WAS INCOMPLETE →→**



If meter does not connect successfully:

- Remove meter from tote
- Cycle tote off and then on
- Re-dock meter

If meter still does not connect:

- Remove meter from tote
- Remove battery from meter
- Replace battery
- Cycle tote off and then on
- Re-dock Meter