

Pathology Volume I – POCT and Phlebotomy/Specimen Collection, POCT WAIVED TESTS POCTW040: Manual Urine Pregnancy Testing – OSOM Card Pregnancy Test Appendix C: Manual Urine Pregnancy Initial Training and Competency Checklist

## Manual Urine hCG Testing - OSOM Card Pregnancy Test Initial Training and Competency

Date:	Unit:								Opera	ator Nam	ie:		JHED ID:	JHED ID:					
POCC or POCT-approv and Competency Chec											-				ectly	performed and each b	oox below is filled out. The	completed	l Training
Competency Category	Performance Indicator											Assessment Tool	Criteria Met	Criteria Not Me iner V					
eLearning	Nev	testir	g personnel	up-to-date	on eLea	rning quiz <i>pri</i>	ior to initi	al traini	ng (MyLe	earning Com	pletion Do	ite:			).		Exam	114	iller v
Reagent Stability and Storage	Mai	Maintains a working knowledge of Test Kits and QC storage:  - Records open and expiration dates on Test Kits and QC vials. Does not use expired reagents.  - Understands expiration dates of consumables (hCG Test Kits – Manufacturer's Expiration) (QC Vials – One Month at Room Temp).															Direct Observation		
QC Performance	Adh	Adheres to QC policies and procedures, including labeling of Test Units:  - Runs appropriate QC at defined frequency (Minimum weekly, opening new kits or QC, unexpected results, training, annually).  - Verifies QC is within acceptable limits, and adheres to defined Corrective Action for out-of-range QC results.  - Properly interprets results and completes the QC Log (Internal QC) (QC Pass/Fail) (Dating) (Recording Results).  - Demonstrates performance of QC, obtains acceptable results, and properly completes QC log (See Below for Initial Competency).															Direct Observation		
Specimen Collection	Stat	States acceptable specimen type (fresh urine), and collection and labeling requirements (no label on lid, two patient identifiers).															Direct Observation		
Routine Patient Testing	Adh	<ul> <li>Adheres to Patient Testing procedure, as outlined in POCTW040:</li> <li>Test Devices labeled properly (patient label with two patient identifiers, or handwritten QC information – L1/Neg or L2/Pos).</li> <li>Maintains a working knowledge of the steps for testing patient samples (uses sample pipette for 3 drops, interpret results at 3 minutes with timer).</li> </ul>															Direct Observation		
Factors Influencing Test Results	Inte	Interprets and evaluates test results in regards to test limitations. Refer to SOP for full list: - Example: bloody urine specimens may impede result interpretation.															Direct Observation		
Documentation of Patient Results		<ul> <li>Confirms acceptability of Internal QC on each Reaction Unit prior to reporting patient results.</li> <li>Uses EPIC code <i>POC99</i> to order and result <i>all</i> Manual Urine hCG Results.</li> </ul>															Direct Observation Review of Records		
Problem Solving Skills	Und	Understands interpretation and corrective action for Invalid and Abnormal Test Results (quantitative testing, repeat test in 48-72 hours). Understands POCT Office Contact Information (5-2645 M-F; POCT Consult on CORUS, POCTGroup email).															Direct Observation		
Safety		Adheres to Standard Precautions throughout testing procedures. Discards biohazardous materials according to policies.															Direct Observation		
Procedure Review	Loca	Acknowledges ongoing competency requirements (Annual <u>successful</u> completion of both levels of QC and passing eLearning module).  Locates and reads the current version of the SOP ( <i>POCTW040</i> ) – accessing through POCT website or HPO.  http://pathology.jhu.edu/department/staff/POCT/procedures.cfm															Direct Observation Operator Acknowledgement	Operator Initials:	
	Date	Time	JHED ID	Test Kit (information on box)		Level 1 QC (Acceptable Result: Neg)				(Ac		Level 2 QC eptable Result: Pos)			C	Comments		•	
		Time		Lot#	Exp Date	Lot#	Exp Date (Room Temp)	Result	Internal QC Ok?	Lot #	Exp Date (Room Temp)	Result	Internal QC Ok?	Pass (√)	Fail (√)	Comments			
Initial Competency																			
I have reviewed thi Is Competer Requires ret	nt to p	erfor	m Manual (	Urine hCG	testin	g using the	osom c	ard Pre	gnancy	Test			tials si	gnify	this	operator:			
Operator Signature:						Date:													
Trainer Signature: _ Revision 6/2020	rainer Signature:evision 6/2020																		