

# Labeling the Sample Vial

## ThinPrep® 5000 Processor

### Proper Labeling



Plastic over wrap is completely removed.



Barcode label: vertical, smooth, aligned with PreservCyt® Solution label

Please avoid placing labels over this blank window on the vial

### Incorrect label placement

can cause a failure to read the barcode or a vial handling error.




Do not...  
- place labels on the bottom of the vial  
- place labels on the vial cap




Avoid...  
- placing multiple labels on top of each other  
- placing the barcode label over patient information  
- wrinkles and peeling  
- placing labels on the torque features of the vial

- If you do label with patient label and cyto order, please make sure they are smooth and flat and do not cover the blank window.
- No tags or label flags

Examples of proper labeling

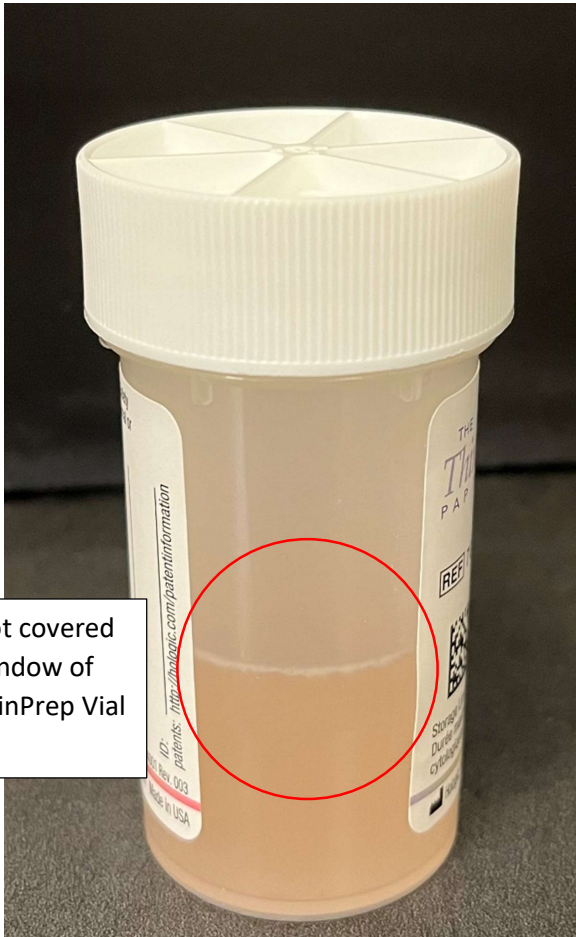

Not covered  
window of  
ThinPrep Vial

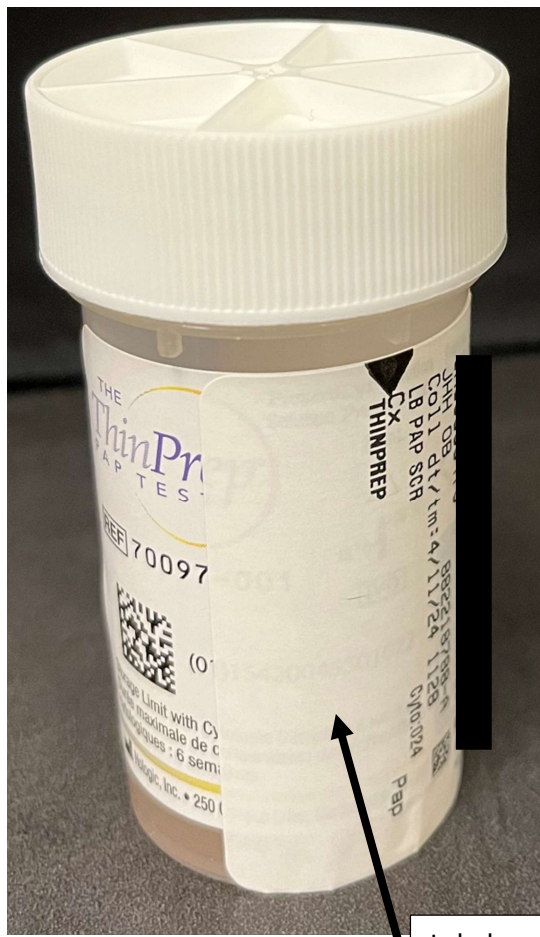

Label can either be  
vertical or horizontal  
as long as it DOES NOT  
cover up the window

