



The Johns Hopkins Hospital  
Department of Pathology  
Point-of-Care Testing Office

## NEAR PATIENT TESTING QUANTIMETRIX DROPPER PLUS URINE CONTROL LOT AND RANGES (Multistix 10SG (Clinitex), Specific Gravity, Pregnancy (OSOM Card Test, and Clinitest))

**NOTE: Control lot effective 12-1-2023**

Level 1 Lot: **204411**

Level 2 Lot: **204412**

Expiration: 4-30-2024

### **Urine Dipsticks Expected Results\* :**

\*Acceptable results determined by Clinitex instrument. If any results are outside of the expected range, the instrument will flag QC result and require corrective action.

### **Pregnancy / Specific Gravity by Refractometer Expected Results:**

	<b>Pregnancy</b>	<b>Specific Gravity</b>
Level 1	Negative	1.018 – 1.024
Level 2	Positive	1.011 – 1.017

**ANY RESULT THAT DOES NOT FALL WITHIN THE EXPECTED RESULTS RANGE MUST BE REPEATED AND CORRECTIVE ACTIONS DOCUMENTED ON THE APPROPRIATE QC LOG SHEET.**

### **Technical Update (239-19-44-01) Issued 08/27/2019:**

*Endogenous crystalline sediment may be observed in the dropper tip. This does not affect the performance of the product.*