



The Johns Hopkins Medical Institutions
The Johns Hopkins Hospital Point-of-Care Testing Program
Urine Pregnancy (hCG) Testing
Test for Operators

Name: _____ Employee ID #: _____

Date: _____ Site: _____ Department: _____

Multiple Choice (circle the correct response):

1. Limitations of the hCG test are:
 - a. Termination of pregnancy
 - b. Trophoblastic disease
 - c. Dilute urine samples with low specific gravity
 - d. All of the above

2. Specimens:
 - a. May be taken any time of the day
 - b. May be refrigerated at time of analysis
 - c. May be collected only in a test tube
 - d. None of the above

3. Quidel QuickVue+ One-Step hCG Combo test kits may be stored:
 - a. At room temperature (15°-30°C)
 - b. Refrigerated at 2°-8°C
 - c. Refrigerated at 10°-12°C
 - d. At room temperature (45°-60°C)

4. The function of the vertical blue line in the small square Control Window is:
 - a. Indicates the Reaction Unit is working properly
 - b. Test has been performed correctly
 - c. May indicate deterioration of reagents if no blue line appears
 - d. All of the above

5. The information required to be documented with the patient's result is:
 - a. Date and time
 - b. The initials of the person who performed the test
 - c. Performance of the Internal Procedural Controls
 - d. Patient's result is noted as a Point-of-Care test.
 - e. All of the above

6. When performing a patient test, _____ drops of urine are dispensed into the Sample Well of the Reaction Unit using a transfer pipette.
 - a. 4
 - b. 3
 - c. 2
 - d. 1

**The Johns Hopkins Medical Institutions
The Johns Hopkins Hospital Point-of-Care Testing Program
Urine Pregnancy (hCG) Testing
Test for Operators**

7. The result is considered Invalid when
- a. No vertical blue line appears in the small square Control Window
 - b. The background color in the large square Read Result Window makes it difficult to interpret the result.
 - c. All of the above
 - d. None of the above.

True or False (use “T” for true and “F” for false):

- _____ 8 The timer is set for 3 minutes and the results are read immediately at 3 minutes for urine.
- _____ 9. Quantimetrix Dropper Plus Controls are run when a box of Quidel QuickVue+ One-Step hCG Combo Reaction Units are initially opened and weekly thereafter on each opened test kit box.
- _____ 10. Reaction Units may be used right from the refrigerator.
- _____ 11. Dispense two (2) drops of control solution into the sample well.
- _____ 12. Any shade of a pink vertical line in the Read Result Window indicates a positive result.
- _____ 13. The (+) and (-) symbols may be used to record patient and control results.
- _____ 14. The “Opened Date” and “New Expiration Date” are to be recorded on the Urine Control bottles when stored at room temperature.
- _____ 15. The Pregnancy Test Kit box does not to be dated when initially opened.
- _____ 16. No result can be reported when the Internal Procedural controls fail.

*© copyright 1999
Johns Hopkins Hospital
All rights reserved*

Revision: 7/10